


U2 Knee *Knee* PS CR
System




Femoral / Tibial Contact

The large 70 / 72 mm consistent radii in the coronal plane provide optimal contact area in the femorotibial articulation and reduce the contact stress of XPE and UHMWPE insert.

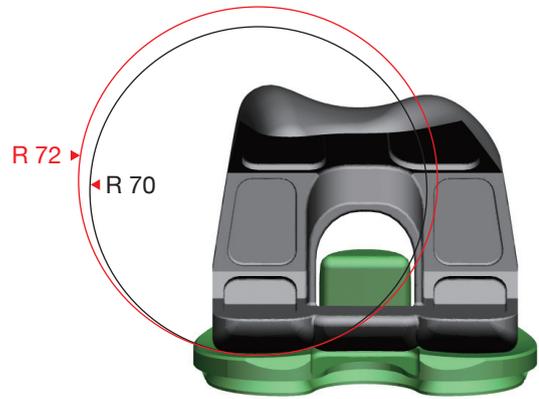


Enhanced Axial Rotation

A matched “round to round” spine and box contact mechanism avoids corner impingement while rotating. The $\pm 12.5^\circ$ I / E rotation conforms to the natural conditions.

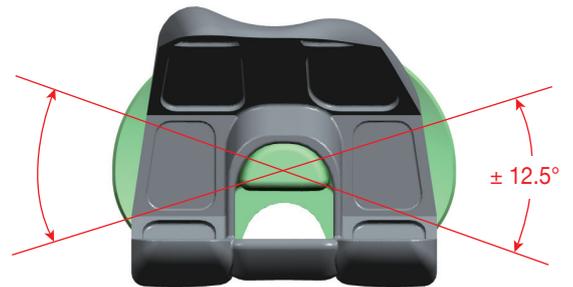

Jumping Distance Safety

A refined spine and cam mechanism minimizes the risk of dislocation at maximum ROM.

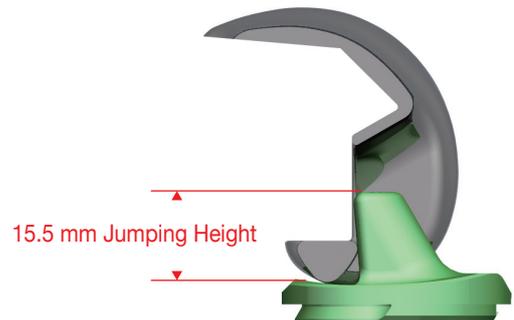


PS Notch Width Consistency

The consistent 18 mm intercondylar width provides full interchangeability between femoral and tibial sizes.




Tibial Insert Design

The built-in 5° posterior slope of the insert enhances the ROM and restores the natural morphology of tibial plateau.


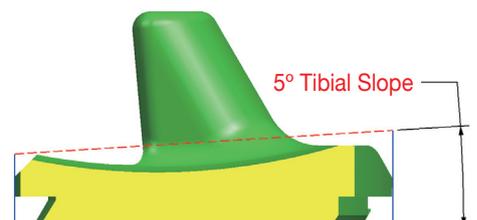


The Spirit of U2 Knee - Preciseness, Safety and Durability

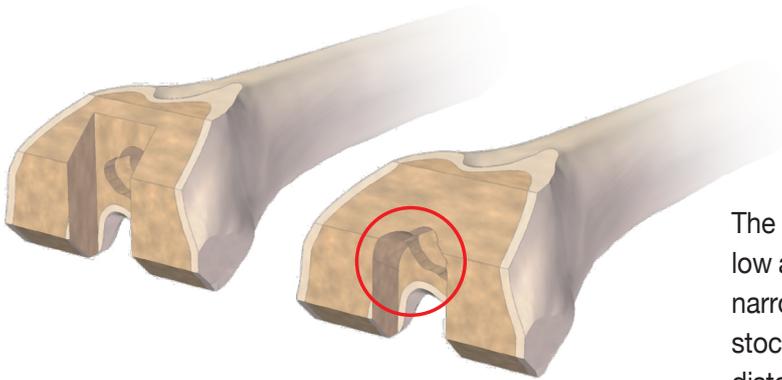

The intercondylar removal volume of U2 PS knee is as low as 5.7 cm³ (size 6). The 30% reduced volume and the narrow 18 mm intercondylar width preserve the bone stock. This design also prevents from the periprosthetic distal condyle fracture.

To prepare the precise box space, the intercondylar notch is made by reaming instead of cutting. The stopper on the reamer minimizes the risk of tibial damage.

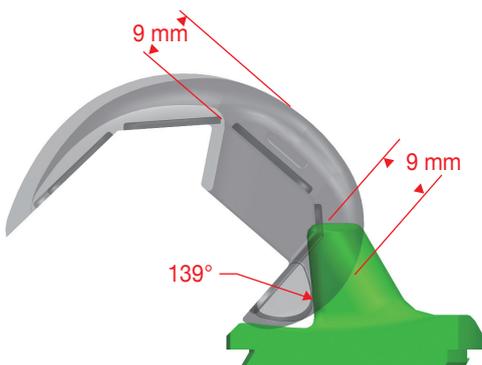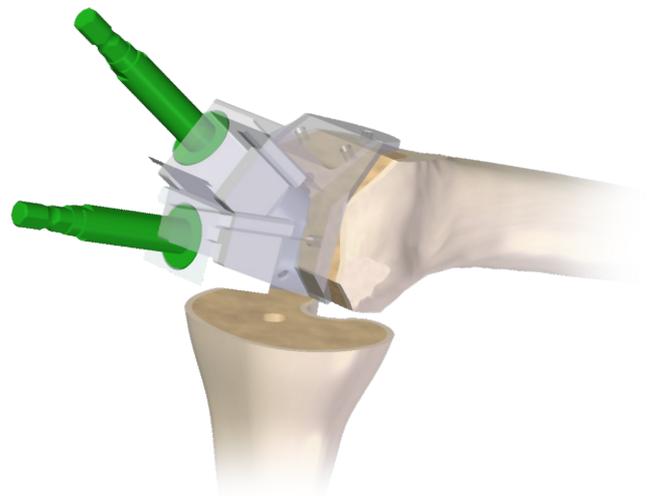

U2 knee achieves 139° ROM with resection thickness of 9 mm in distal and posterior condyles. The ROM fulfills daily activities of patients with high standard of motion safety.

PS

Femoral Component

CoCrMo alloy, 7 sizes

Size	1	2	3	4	5	6	*7
AP(mm)	52	56	60	64	68	72	76
ML(mm)	56	60	64	68	72	76	80

* PS Only

Tibial Insert


UHMWPE & XPE

5 thicknesses (9,11,13,15,18 mm)

Tibial Baseplate

Titanium alloy, 6 sizes

Size	1	2	3	4	5	6
AP(mm)	42	44.5	47	49.5	52.5	55.5
ML(mm)	63	66	69	72	76	80

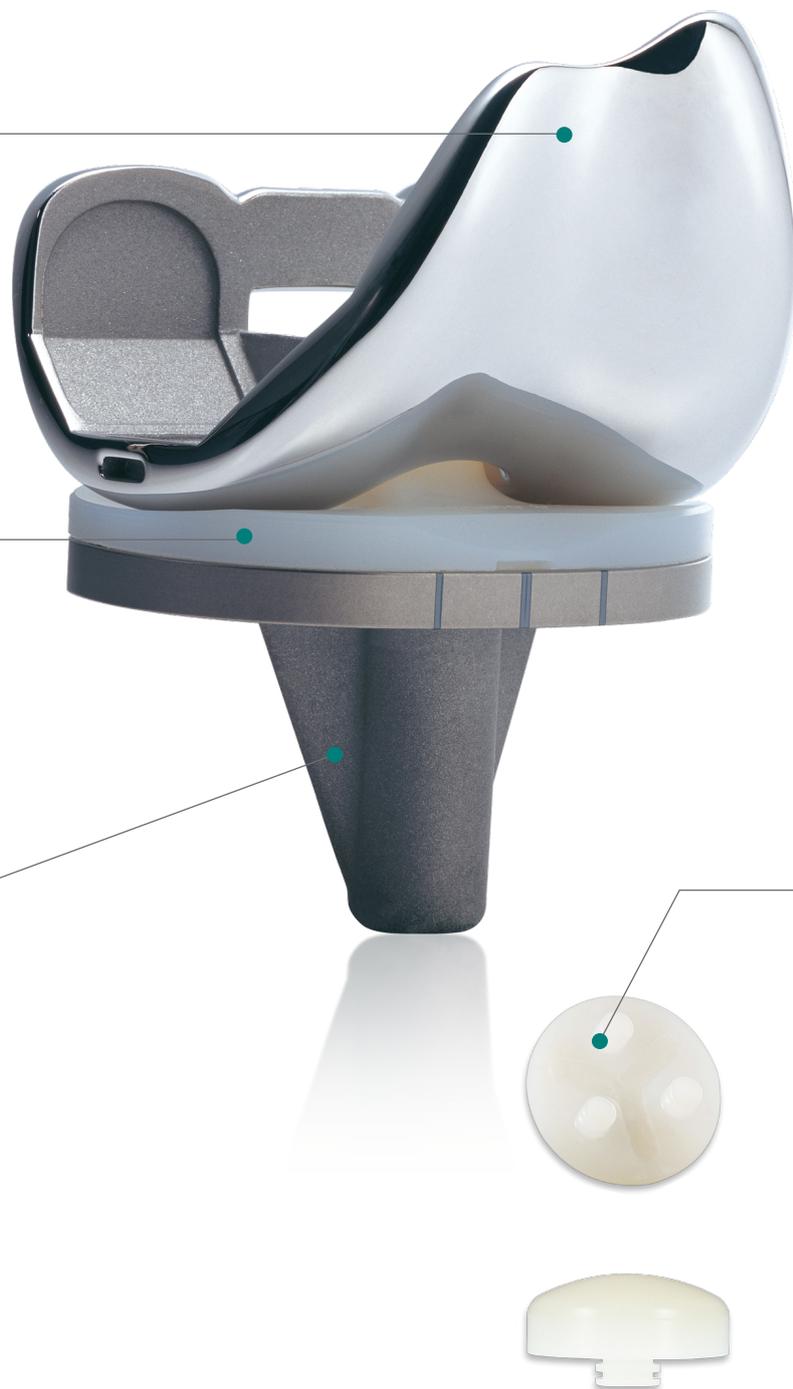



CR

XPE System

Patellar Component

UHMWPE & XPE


Onset, 3 pegs, 7 sizes

Size	XS	S	M	L	XL	2XL	3XL
Diameter(mm)	26	29	32	35	38	41	44
Thickness(mm)	7	8	8.5	9	9.5	10	10.5

Inset, 1 peg, 4 sizes

Size	S	M	L	XL
Diameter(mm)	22	25	28	32
Thickness(mm)	8	10	10	10

Wear Resistance of XPE versus Traditional UHMWPE

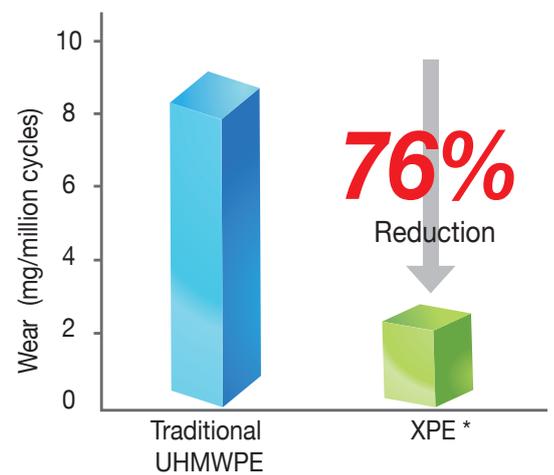

* Report Number: UOC-RD-TR-10028



Office

■ Taiwan

12F, No. 80, Sec. 1, Chenggong Rd, Yonghe Dist.,
New Taipei City 23452, Taiwan
Tel: +886 2 2929-4567 Fax: +886 2 2922-4567

■ USA

UOC USA Inc.

20 Fairbanks, Suite 173
Irvine, CA 92618

Tel: +1 949-328-3366 Fax: +1 949-328-3367

■ China

United Medical Instrument Co. Ltd.

7F, No.1405, GongheXin Road, Shanghai, China
Tel: +86 21 3661-8055 Fax: +86 21 6628-3031

■ EU Representative

mdi Europa GmbH

Langenhagener Strasse 71, 30855 Langenhagen, Germany

Tel: +49-511-3908 9530 Fax: +49-511-3908 9539



All Rights Reserved, © 2013 United Orthopedic Corp.
www.uoc.com.tw

0188-B-02-02 Ver.3